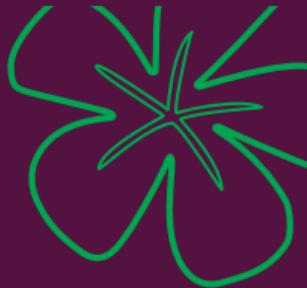
SUNSEPT

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EN SPRAY AND WIPE DISINFECTANT

Ready-for-use alcoholic fast disinfectant for the
disinfection of small alcohol-resistant areas.

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EN Type of application: EEC/93/42: Ready-for-use alcoholic quick disinfectant for the prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for the final (terminal) disinfection of invasive medical devices. As per the Biocidal Products Directive: Ready-for-use alcoholic quick disinfectant for the disinfection of small alcohol-resistant surfaces. **Notes on application:** For disinfection, moisten a suitable cloth with the disinfectant solution and wipe the area, or apply the solution directly to the surface and then wipe. For the spray disinfection, spray the disinfectant onto the area or objects to be disinfected and then wipe. Make sure that the surface to be disinfected is fully covered with liquid. Then let the disinfectant take effect. The disinfectant is not suitable for use on materials that are not alcohol-resistant (e.g. acrylic glass). This disinfectant solution is only allowed to be used to soak wipers which are compatible with this solution (according to EN 1615). **Application times:** Bacteria and candida albicans (VH, without mechanisms): Conc. 30 sec (clean) Conc. 1 min. (dirty) Bacteria and candida albicans (EN 13727, EN 13624, dirty): Conc. 30 sec Bacteria and candida albicans (EN 1276, EN 1650, dirty): Conc. 1 min. TbB (tuberculosis) inactivation (according to DGHM, EN 14348): Conc. 30 sec. (clean) Conc. 1 min. Rota viruses (EN 14476, dirty): Conc. 1 min. Rota viruses (EN 14476, dirty): Conc. 30 sec. Limited viruzidal Plus: Conc. 1 min. Viruzid (low-level) according to DVV 2012 (carrier test, clean): Conc. 3 min. Application times for wipe-systems: Bacteria and candida albicans according to EN 1615, VH (with mechanisms), dirty. Conc. 1 min. Norovirus (EN 14476, clean): Conc. 3 min. 2miplication times for wipe-systems: Bacteria and candida albicans according to EN 1615, VH (with mechanisms), dirty. Conc. 1 min. 318 Causes serious eye damage. Flammable liquid and vapour. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear eye protection / face protection. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container in accordance with local/regional/national/international regulations. Flammable! Do not spray into naked flames! Flash point as per DIN 51755: approx. 25°C. Application max. 50ml per m2. The total applied amount of disinfectant per room must not exceed more than 100ml per m2 of room area. Keep the containers tightly sealed. Only put the container into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99.

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PRISMAN GmbH