
SUNSEPT

SUN402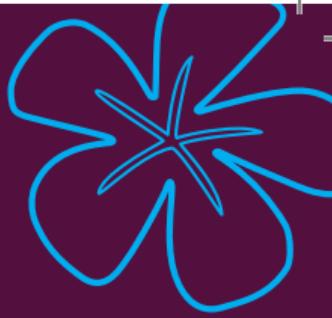

Instrument Disinfectant plus conc.

EN INSTRUMENT DISINFECTANT PLUS CONCENTRATE

Product for manual and terminal disinfection of non-critical and semi-critical medical and surgical instruments in the dipping bath process.

- ✓ Bactericidal, levurocidal, tuberculocidal, virucidal, fungicidal
- ✓ Non corrosive
- ✓ Aldehyde-free

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E Type of application: Preparation for the manual and terminal disinfection of unicritical and semi-critical medical and surgical instruments using an immersion bath procedure. Not suitable for the terminal disinfection of critical medical devices! Instructions for use: Dilute the concentrate (use dispensing bottles or dispensing equipment) with 10 °C to 30 °C warm water (at least drinking water quality). Before disinfection, the RKI (Robert Koch Institute) / BfK/M (Federal Institute for Drugs and Medical Devices) recommends a thorough pre-cleaning or mechanical cleaning and, if necessary, intermediate rinsing of the instruments. Afterwards, immediately cleanse the cleaned and opened instruments in the working solution. All surfaces and cavities to be disinfected must be fully covered by the working solution. Depending on the degree of contamination, the working solution may need to be renewed (RKI recommendation: at least once per working day). After the disinfection time, rinse the instruments thoroughly, ideally with demineralised water, and dry them. The RKI recommends a final sterilisation for critical instruments. Do not mix with cleaning agents or other disinfectants. The reprocessing instructions of the instrument manufacturers and the reprocessing instructions of the RNUKRINNO recommendation are to be observed! When using continuously material changes of polycarbonate and polyamide cannot be ruled out; however, they can be largely avoided by rinsing them sufficiently after each application. When using new materials, check for compatibility in advance. The working solution can be disposed of via the wastewater. To ensure effectiveness, the dosage quantities and application times must be observed as closely as possible and not be undercut. Avoid overuses for reasons of environmental protection and material compatibility. Suitable for use in ultrasonic baths. Suitable for (dental) practices, hospitals, retirement homes, care and rescue services as well as pediatric studios. Application times: Bacteria and Candida albicans (EN 13727, EN 14561, 14562 and VRH), dirty conditions: 2.0 % 15 min., 1.0 % 60 min., TB (M. terrae) as per EN 14348, EN 14563 (clean conditions); 2.0 % 60 min., Mycobacteria as per EN 14348, EN 14563 (clean conditions); 3.0 % 60 min., "Limited virucidal" enveloped viruses (e.g. HBV, HCV, HIV) as per RKUDWV and EN 17111, dirty conditions: 3.0 % 15 min., Norovirus as per EN 14476 and EN 17111 (clean conditions); 3.0 % 60 min., 4.0 % 30 min., Polyomaviruses as per EN 14476 and EN 17111 (clean conditions); 3.0 % 60 min., 4.0 % 30 min., Fungi (A. brasiliensis) as per EN 13624, EN 14562 (clean conditions); 3.0 % 60 min., 4.0 % 30 min., Adenovirus as per EN 14476 and EN 17111 (clean conditions); 3.0 % 60 min., 4.0 % 30 min., Poliovirus as per EN 14476 and EN 17111 (clean conditions); 3.0 % 60 min., 4.0 % 30 min., Eye protection: If ON SKIN or hair: Take off immediately all contaminated clothing. Rinse skin with shower/water IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. Contains dodecyltrimethylammonium chloride, 1-aminopropan-2-ol, N,N-dodecylpropan-1,3-diamine. Ingredients as per 64/2004/EC; (EC) 64/2004/CE. Contains 5-15% non-ionic surfactants, 5-15% cationic surfactants, fragrances. Notes on incidents: All serious incidents related to the device shall be reported by the user/participant to the manufacturer and to the competent authority of the Member State in which the user/participant is established.

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■ PRISMAN GmbH