# SUNSEPT

## ABSAUGANLAGENREINIGER KONZ.

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**EN** **SUCTION SYSTEM CLEANER**  
For the cleaning of dental suction systems  
incl. their drainageconduits.

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**EN** **Type of application:** Foam-inhibited cleaning concentrate for the cleaning of dental suction  
systems incl. their drainage conduits. Regular usage prevents the formation of deposits and  
blockages or dissolves already existing deposits. **Instructions for usage:** The special cleaner is used  
as a 10% solution for problem cases (100ml/l water). The residence time is 1-2 hours. With regular  
usage the cleaner is used as a 5% solution (50ml/l water) 1-2 times a week with a residence time of  
30min. 2 liter of the usage solution is sucked through the suction tubes and rinsed with one litre of  
water through the system following the residence time (This corresponds 2 x 1/2 dosage cap in 2 liter  
water). **Safety notes:** H319 - Causes serious eye irritation. Wash thoroughly after handling. Wear pro-  
tectice gloves and eye protection. If in EYES: Rinse cautiously with water for several minutes. Remove  
contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice  
/ attention. 648/2004/EG: <5% amphotere Tenside, <5% Phosphonate.

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