# SUNSEPT

## SUN500

### Suction System Disinfection conc.

**EN SUCTION SYSTEM DISINFECTION CONCENTRATE**

For disinfecting and cleaning dental suction systems and spittoons.

- ✓ Aldehyde-free
- ✓ Very high material compatibility
- ✓ Low-foaming

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**REF** SUN500K

**2,5L e**

**EN** Areas of application: High performance solution designed for cleaning and disinfecting of dental Evacuation systems (incl. suction hoses and amalgam separator) and spittoons. Indications for use: The concentrate must be first diluted with water. Recommended application concentration: 2%. Add 20 ml concentrate with tap water to 1.000 ml and aspirate the solution through the suction hoses. Regularly, the suction system should be disinfected once or twice a day. The spittoon is disinfected by puring approx. 250 ml of the ready-to-use dilution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately. Effectivity: Bacteria and *Candida albicans* in accordance to VAH as instrument disinfection (high contamination): 2,0% 5min. / 1,0% 60min. Bacteria in accordance to EN 13727 and EN 14561 (high contamination): 3,0% 2min. *Candida albicans* in accordance to EN 13624 and EN 14562 (high contamination): 3,0% 2 min. Enveloped viruses (i.e. HBV, HIV, HCV) in accordance to RKI/DVV (with pollution): 2,0% 2min. Safety Instruction: H 314 Causes severe skin burns and eye damage. H 411 Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains N,N-Dimethyl-N-Diethylammoniumchlorid, Kaliumhydroxid. REG 648/2004/EC: < 5% nonionic surfactants, < 5% cationic surfactants, fragrances.

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**PRISMAN GmbH**